

DOMINO

PEOPLE

Domino People enables to transport manual and electronic wheelchairs up to 400 kg. It is particularly suitable for public buildings such as municipalities, schools, hospitals, but also retirement homes, banks and condominiums.



DOMINO
PEOPLE



DOMINO

PEOPLE

Thanks to the "Automatic" load management technology, the wheelchair is automatically tilted on the stairs, thus making transport easier and safer. In addition, the tracks are suitable for both indoor and outdoor surfaces, so it is possible to travel up and down stairs made of virtually any material, even with irregular steps and of different height.



No restriction on the height of the steps, since it is possible to travel even on irregular stairs.




DOMINO

PEOPLE

Thanks to the independent tracks and the advanced electronics it is also possible to go up and down spiral staircases.

The operator only needs to correct the trajectory by moving the joystick so as to adapt to the type of stairs in question.

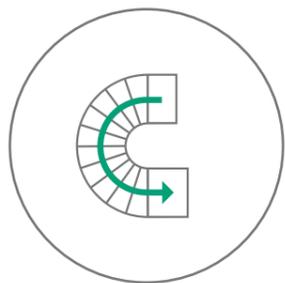

The independent tracks with double electric motor guarantee maximum manoeuvring comfort even in narrow spaces.



DOMINO

PEOPLE

It is possible to correct at any time the safety angle of the wheelchair preset by the electronics. This may be necessary to tackle particularly steep stairs or particularly heavy wheelchairs.

The control for the correct positioning of the wheelchair, both for Domino People and Domino People Slim, is always the operator's first task before continuing the journey on the stairs, when going up and down.



Simply press the "M" button on the control to set a new inclination of the wheelchair






Maximum manoeuvrability.

DOMINO

PEOPLE

Domino People gives the freedom to move on several types of staircases, both indoors and outdoors, whether straight or spiral, without ever having to get out of the wheelchair.

It is pushed from one step to another with puncture-resistant tyres and it adapts perfectly to steps of any height and shape.

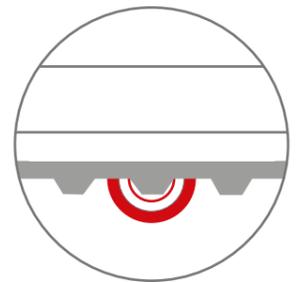



Wheelchair always well-balanced and safe.



DOMINO

PEOPLE

Feeling safe during transport on stairs also allows you to enjoy the most pleasant moments of the ride. Security and reliability are key components of Domino People and Domino People Slim

TECHNICAL FEATURES



Domino People has telescopic ramps to facilitate access of all types of wheelchair.



Access to the load surface must always be in reverse.




The telescopic ramps are equipped with mechanical supports in the middle, in order to support heavy loads and have excellent durability over time.



Secure the wheelchair using the supplied straps and store the telescopic ramps in the appropriate housings.



It is possible to order a load surface with customised dimensions to better adapt to specific wheelchairs.

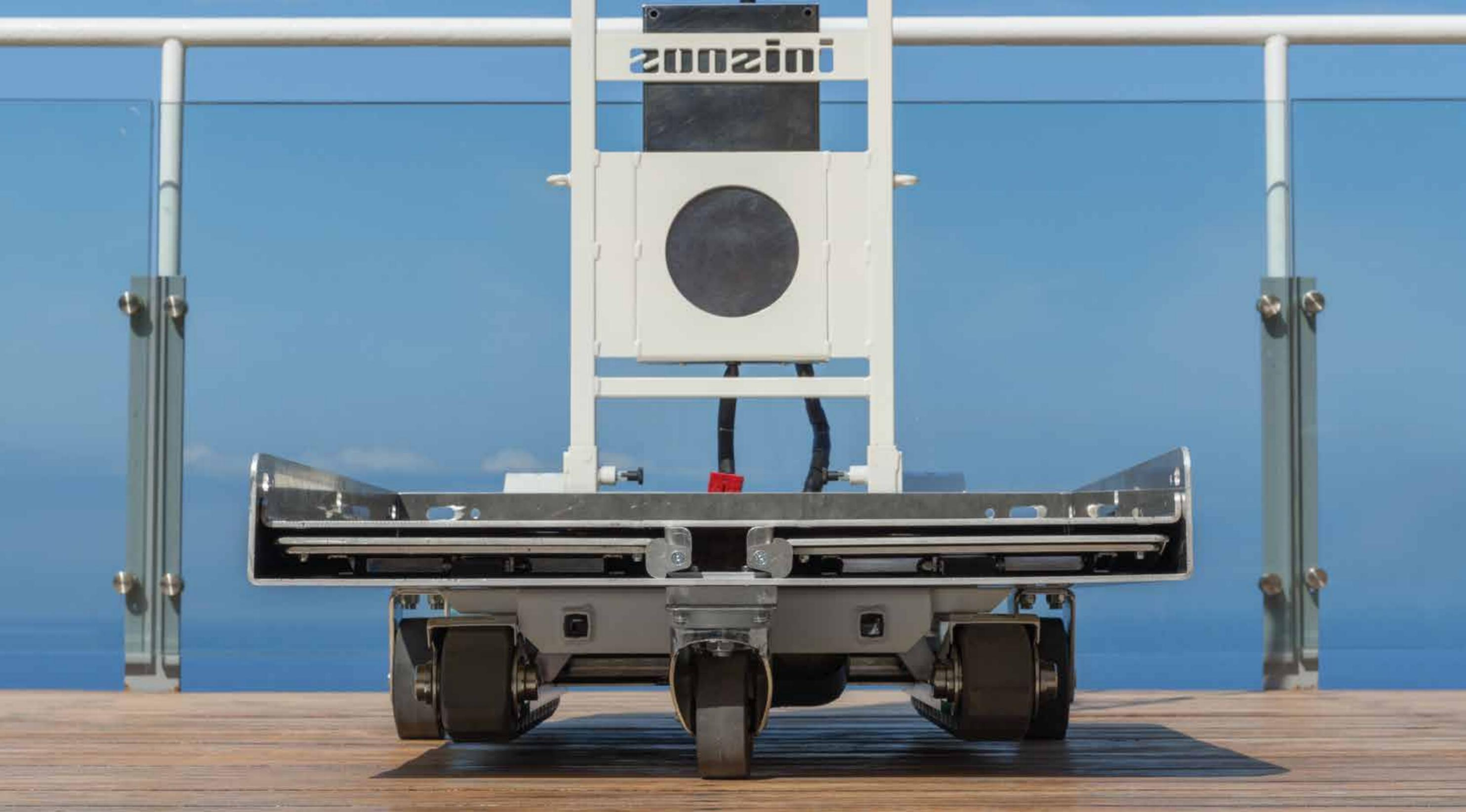

When not working on stairs with the tracks, the three wheels with which Domino People and Domino People Slim are equipped always rest on the ground to facilitate movement from one step to another.